


**To obtain a Specimen
Collection Kit contact:
WAPHL Mailroom
206-418-5579**

**Department of Health
Public Health Laboratories
1610 NE 150th Street
Shoreline, WA 98155**



Enterics
(Stool)

**Specimen
Collection Kit**

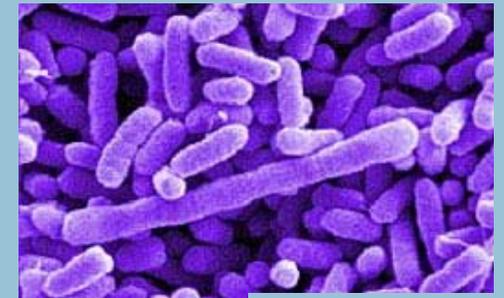

Salmonella

PUBLIC HEALTH
ALWAYS WORKING FOR A SAFER AND
HEALTHIER WASHINGTON

Enterics Collection Kit

Shipping to the Public Health Laboratories

Instructions:

Collect the specimen with the swab provided (equipped with red cap). Twist off the white cap from the Transport Media tube and discard. Place swab into the transport tube pushing it in to engage the red cap and thus sealing the tube.

*

Package and label according to the Department of Transportation regulations for Diagnostic Specimens.

*

Place completed shipping container into the Lab Pak bag for USPS mailing purposes and ship at ambient temperature to the WAPHL as soon as possible.



Collection Kit Contents:

- 1 Shipping container³
- 1 Cary-Blair transport media/swab¹
- 1 Lab Pak bag⁴
- 1 lab form²

Transport Packages to:

Washington Public Health
Laboratories
Attention: Enteric
Bacteriology
1610 NE 150th Street
Shoreline, WA 98155



Phone: 206-418-5400

Fax: 206-418-5445