

## Specimen Collection and Submission Instructions

### *Neisseria* species Identification (Version 1)

Page 1 of 2

Specimen Type	Collection Time	Collection Frequency	Collection Procedures	Transport Media	Shipping & Handling (S&H)**
Bacterial isolate	NA	NA	NA	Chocolate agar	<ul style="list-style-type: none"> <li>• <b>Transport device:</b> Chocolate agar plate or tube.</li> <li>• <b>Rejection Criteria:</b> <ul style="list-style-type: none"> <li>○ <i>N. meningitidis</i> collected from non-sterile sites.</li> <li>○ Mixed culture.</li> <li>○ Non-viable specimen.</li> <li>○ Broken transport device.</li> <li>○ Leaking specimen.</li> </ul> </li> <li>• <b>Transport:</b> Ship at ambient temperature. Ship as Category B. <i>N. gonorrhoeae</i> isolates lose viability quickly in ambient air. Consider using a CO<sub>2</sub> gas pack when shipping.</li> <li>• <b>Storage:</b> Store at ambient temperature.</li> </ul>

**General Rejection Criteria (for additional details, see S&H)**

- Failure to follow specific S&H requirements.

NOTE:

Specimens will not be processed until ALL of the following information are provided on requisition form:

- Patient name, patient's county of residence, and second identifier.
- Two matching identifiers on both the tube and the laboratory requisition form.
- Specimen type, date of collection, and test requested.
- Submitter name, address, and telephone/FAX numbers.
- Reason for testing at WAPHL.

**Collection kit availability**

Not available.

**Comments**